
Phonak Roger SoundField – Quick Guide

Roger SoundField: Combined performance

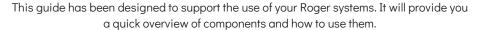




Activate the Speaker and Microphones: Speaker: Connect power supply to the speaker and turn röger via the switch located at the bottom of the speaker. DSF Speaker: On/Off Switch Power Cable -Indicator light Meaning PHONAK TSM off, out of range or in small group mode TSM is connected and ready for operation

Touchscreen Microphone (TSM):

Before getting started, fully charge microphones. Note: (It take approx. 2 hours to fully charge microphones)

- 1. Turn on TSM
- 2. Place the TSM around neck, the distance from the mouth to the top of the TSM should be around 20cm.

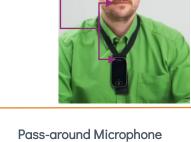

Student Microphone



TSM muted



